

Category: Biochemistry

Title: Estimation of ALT (Alanine Aminotransferase) in blood serum samples using minichem ARK Diagnostics

SOP No.: Biochemistry02/01

Date first effective: 01stJanuary2025

Review date: 31stDecember 2025

Department of Clinical Pharmacology, 1st Floor, New MS Building,
Seth GS Medical College & KEM Hospital, Parel, Mumbai 400012

Category: Biochemistry

Title: Estimation of ALT in blood serum samples using mini chem ARK diagnostics

SOP No: Biochemistry02/01

Total pages: 4

Date first effective: 01stJanuary2025

Next Review date: 31stDecember 2025

Author: Vaishali Hadikar
Laboratory Technician

Signature with date:

V Hadikar
31/Dec/24

Department of Clinical Pharmacology
Seth GS Medical College & KEM Hospital,
Parel, Mumbai - 400 012.

Lab In charge: Dr. Sheetal Kudtarkar
Project Scientist

Signature with date:

Sheetal Kudtarkar
31/Dec/24

Reviewer: Dr. Mahesh Belhekar
Associate Professor

Signature with date

Bel
31/Dec/24

Dr. Mahesh N. Belhekar
Associate Professor
Department of Clinical Pharmacology
New MS Building, First Floor,
Seth GS Medical College and KEM Hospital
Acharya Donde Marg, Parel,
Mumbai - 400 012. India

Approved by: Dr. Nithya Gogtay
Professor and Head

Signature with date:

Nithya Gogtay
31/12/24
Dr. Nithya Gogtay
Professor & Head
Department of Clinical Pharmacology
1st Floor, MS Building,
Seth GS Medical College & KEM Hospital
Parel, Mumbai - 400 012.

Category: Biochemistry

Title: Estimation of ALT (Alanine Aminotransferase) in blood serum samples using minichem ARK Diagnostics

SOP No.: Biochemistry02/01

Date first effective: 01st January 2025

Review date: 31st December 2025

Department of Clinical Pharmacology, 1st Floor, New MS Building,
Seth GS Medical College & KEM Hospital, Parel, Mumbai 400012

Table of contents

No.	Contents	Page No.
1	Purpose	3
2	Scope	3
3	Responsibility	3
4	References to other applicable SOPs	3
5	Detailed Instructions	3

Category: Biochemistry

Title: Estimation of ALT (Alanine Aminotransferase) in blood serum samples using minichem ARK Diagnostics

SOP No.: Biochemistry02/01

Date first effective: 01stJanuary2025

Review date: 31stDecember 2025

Department of Clinical Pharmacology, 1st Floor, New MS Building,
Seth GS Medical College & KEM Hospital, Parel, Mumbai 400012

1. Purpose

The purpose of this Standard Operating Procedure (SOP) is to outline the procedure for estimation of ALT on minichem ARK Diagnostics in the Biochemistry lab of Department of Clinical Pharmacology.

2. Scope

This SOP covers the procedure of estimation of ALT on minichem ARK Diagnostics using blood serum.

3. Responsibility

Lab technician, Lab attendant, or any other appropriately qualified staff in the team, designated by the Head of Department, will be responsible for analysis.

4. Reference

- Departmental SOP no 10/05: Blood collection and sample processing
- Departmental SOP no 24/02: Waste management.
- Biochemistry labs SOP no 01/01: minichem ARK Diagnostics
- Kit inserts

5. Detailed Instructions

1. The whole blood sample is collected and processed as per SOP no. 10/05.
2. Remove the QCA reagent kit kept in the **Refrigerator**[between 2-8 degree Celsius] located in the main Biochemistry laboratory of Department of Clinical Pharmacology, M.S. Building, 1st Floor. Verify the expiry date before using the kit.

Use kit: Spin react ALT Kit,

3. As per SOP no. 01/01 for Biochemistry analysis on mini chem ARK Diagnostics, select the parameter 'ALT' and set the machine for measurement of the sample.

Category: Biochemistry

Title: Estimation of ALT (Alanine Aminotransferase) in blood serum samples using minichem ARK Diagnostics

SOP No.: Biochemistry02/01

Date first effective: 01stJanuary2025

Review date: 31stDecember 2025

Department of Clinical Pharmacology, 1st Floor, New MS Building,
Seth GS Medical College & KEM Hospital, Parel, Mumbai 400012

4. As per the reagent kit insert, to 50 μ l of the sample add 500 μ l of reagent 1 and 100 μ l of reagent 2.
5. After addition of the reagents, immediately aspirate the test solution through the sipping tube by pressing the button behind the tube.
6. The machine will give a beep sound and will continue with the measurement of the concentration for a period of 120 seconds.
7. After the delay time of 120 seconds, the machine will beep and will display the absorbance followed by the reading.
8. Values are entered in the report, evaluated and signed by the biochemist, laboratory in charge and the PI.
9. Once the ALT parameter is completed for all samples, aspirate distilled water for washing purpose.
10. After washing is completed, switch off the adaptor and then switch off the main switch.
11. Samples are discarded as per departmental SOP no. 24/02 (SOP of waste management).