

CURRENT

Category: Biochemistry

Title: Estimation of Bilirubin in blood serum samples on minichem ARK diagnostics

SOP No.: Biochemistry06/02

Date first effective: 01st January 2025

Review date: 31st December 2025

Department of Clinical Pharmacology, 1st Floor, New MS Building,
Seth GS Medical College & KEM Hospital, Parel, Mumbai 400012

Category: Biochemistry

Title: Estimation of bilirubin in blood serum samples on minichem ARK Diagnostics

SOP No: Biochemistry06/02

Total pages: 4

Date first effect: 01st January 2025

Next Review date: 31st December 2025

Author:

Vaishali Hadikar
Laboratory Technician

Signature with date:

V Hadikar
31/Dec/2024

Department of Clinical Pharmacology
Seth GS Medical College & KEM Hospital,
Parel, Mumbai - 400 012.

Lab In charge:

Dr. Sheetal Kudtarkar
Project Scientist

Signature with date:

Sheetal Kudtarkar
31/Dec/24

Reviewer:

Dr. Mahesh Belhekar
Associate Professor

Signature with date

pell
31/Dec/2024
Dr. Mahesh N. Belhekar
Associate Professor
Department of Clinical Pharmacology
New MS Building, First Floor,
Seth GS Medical College and KEM Hospital
Acharya Donde Marg, Parel,
Mumbai - 400 012. India

Approved by:

Dr. Nithya Gogtay
Professor and Head

Signature with date:

y
31/12-24
Dr. Nithya Gogtay
Professor & Head
Department of Clinical Pharmacology
1st Floor, MS Building,
Seth GS Medical College & KEM Hospital
Parel, Mumbai - 400 012.

Category: Biochemistry

Title: Estimation of Bilirubin in blood serum samples on minichem ARK diagnostics

SOP No.: Biochemistry06/02

Date first effective: 01st January 2025

Review date: 31st December 2025

Department of Clinical Pharmacology, 1st Floor, New MS Building,
Seth GS Medical College & KEM Hospital, Parel, Mumbai 400012

Table of contents

| No. | Contents | Page No. |
|-----|-------------------------------------|----------|
| 1 | Purpose | 3 |
| 2 | Scope | 3 |
| 3 | Responsibility | 3 |
| 4 | References to other applicable SOPs | 3 |
| 5 | Detailed Instructions | 3 |

Category: Biochemistry

Title: Estimation of Bilirubin in blood serum samples on minichem ARK diagnostics

SOP No.: Biochemistry06/02

Date first effective: 01stJanuary2025

Review date: 31st December 2025

Department of Clinical Pharmacology, 1st Floor, New MS Building,
Seth GS Medical College & KEM Hospital, Parel, Mumbai 400012

1. Purpose

The purpose of this Standard Operating Procedure (SOP) is to outline the procedure for estimation of bilirubin (total & direct) on minichem ARK Diagnostics in the Biochemistry lab of Department of Clinical Pharmacology

2. Scope

This SOP covers the procedure for estimation of bilirubin (total & direct) on minichem ARK diagnostics in blood serum.

3. Responsibility

Lab technician, Lab attendant, or any other appropriately qualified staff in the team, designated by the Head of Department, will be responsible for analysis.

4. Reference

- Departmental SOP no 10/05: Blood collection.
- Departmental SOP no 24/04: Waste management.
- Biochemistry lab SOP no 1/01: Analysis using minichem ARK instrument.
- Kit inserts

5. Detailed Instructions

1. The whole blood sample is collected and processed as per SOP no. 10/05.

Remove the QCA Jendrassik Grof method reagent kit kept in the **Refrigerator** [between 2-8 degree Celsius] located in the main Biochemistry laboratory of Department of Clinical Pharmacology, M.S. Building, 1st Floor. Verify the expiry date before using the kit.

Use kit: Spin react BilirubinKit [JendrassikGrof Method]

2. As per SOP no. 10/03 for Biochemistry analysis on minichem ARK Diagnostics, select the parameter 'TOTAL BILIRUBIN' and set the machine for measurement of the sample.

CURRENT

Category: Biochemistry

Title: Estimation of Bilirubin in blood serum samples on minichem ARK diagnostics

SOP No.: Biochemistry06/02

Date first effective: 01st January 2025

Review date: 31st December 2025

Department of Clinical Pharmacology, 1st Floor, New MS Building,

Seth GS Medical College & KEM Hospital, Parel, Mumbai 400012

3. As per the reagent kit insert, to 40 μ l of the sample add 400 μ l of reagent 1, 100 μ l of reagent 2 and 10 μ l of reagent 3.
4. After addition of the reagents, immediately aspirate the test solution through the sipping tube by pressing the button behind the tube.
5. The machine will give a beep sound and will continue with the measurement of the concentration for a period of 30 seconds.
6. After the delay time of 30 seconds, the machine will beep and will display the absorbance followed by the reading.
7. Values are entered in the report, evaluated and signed by the biochemist/laboratory in charge and Study-coordinator and PI.
8. Once the total bilirubin parameter is completed for all samples, aspirate distilled water for washing purpose.
9. After washing is completed, switch off the adaptor and then switch off the main switch.
10. Samples are discarded as per departmental SOP no. 24/02 (SOP of waste management).